



SPECIMEN COLLECTION

Nasopharyngeal Swab

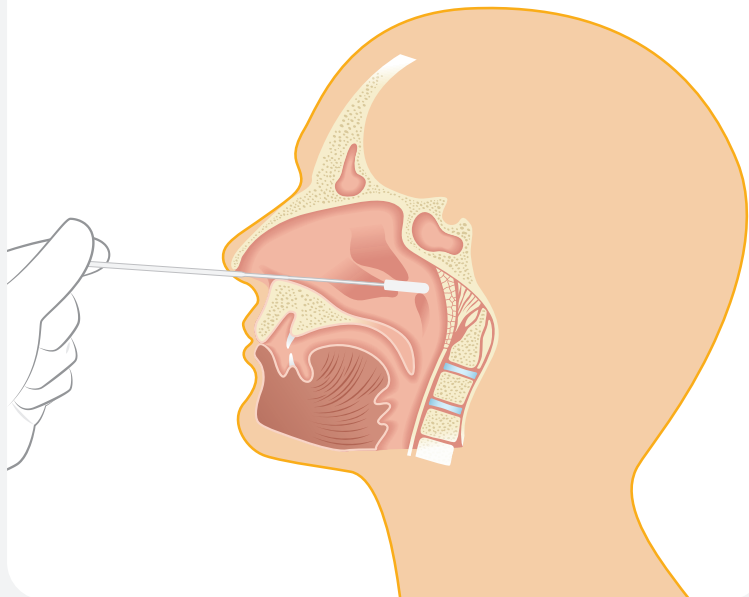


Sputum


1 Rinse your mouth with water

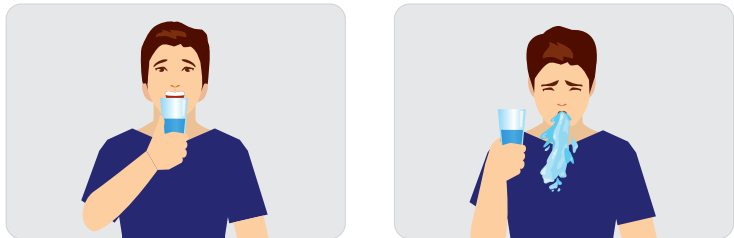

2 Breath in and out 3 times

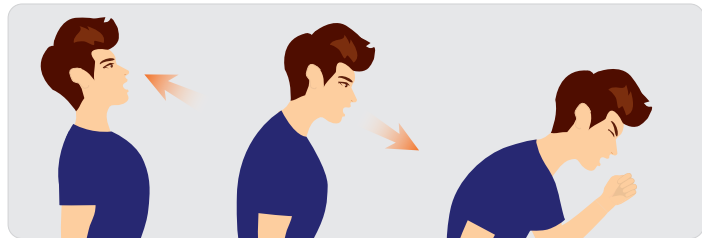

3 Give a sputum sample




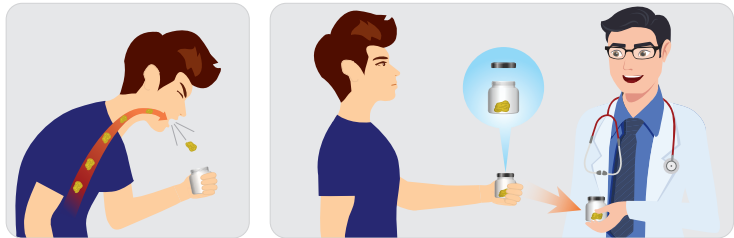


4



Put the swab into the container containing the sputum, turn it several times and soak it sufficiently

TEST PROCEDURE

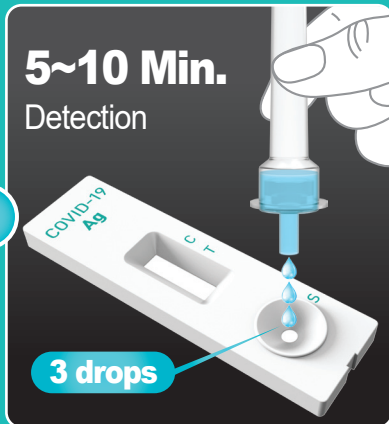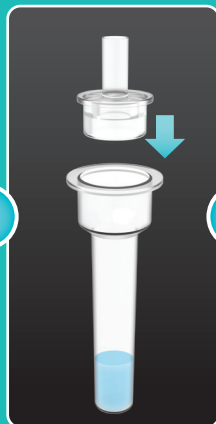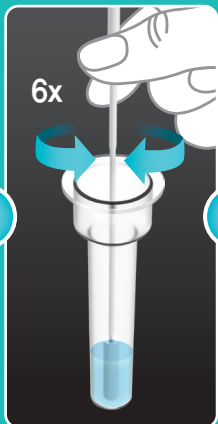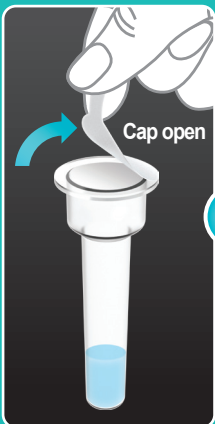


5~10 Min.
Detection

3 drops

5 STEP

Add 3 drops of the extraction solution with specimen into the sample well. After 10 minutes, interpret the test results.

INTERPRETATION OF THE RESULTS

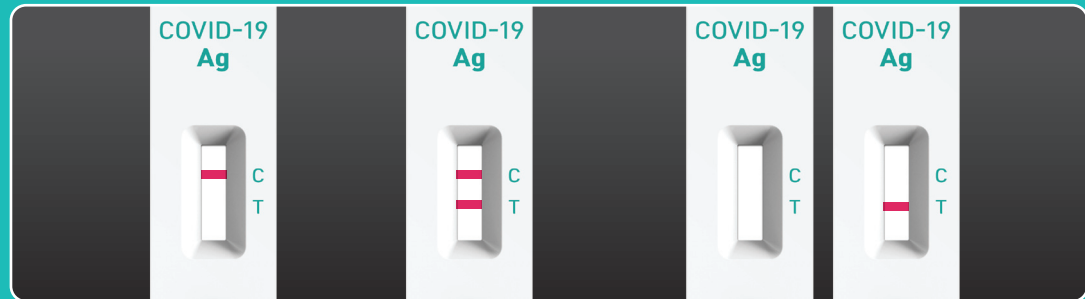


Negative

Positive

Invalid

Positive Result

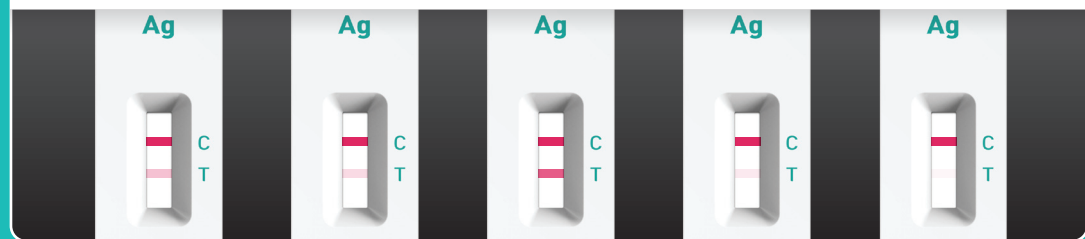

If there is any blurry line or color change on the T line within 15 minutes, it should be read as positive.