

iONE Neutralized Antibody Detection Kit

For in vitro qualitative detection of human IgG antibodies against SARS-CoV-2 and neutralizing antibodies



▶ Video Introduction on iONE Neutralized Antibody Detection Kit




Product Features


- Results are available in **25~30 minutes**
- High Throughput (**Over 10,000 tests/day**)
- **Cost-effective** and **large-scale** solution

Simple Test Step


1. Add specimen into sample well.
2. Add diluent (100uL/3drops) into sample well.

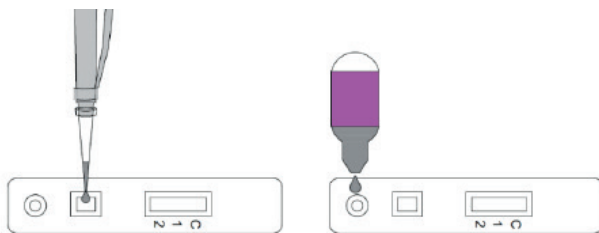


Result Analysis

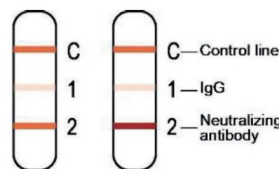


- IgG Positive and Neutralizing antibody **Negative**
→ Three lines appear

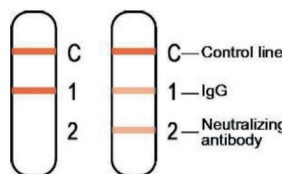

- IgG Positive and Neutralizing antibody **Positive**
→ Two or three lines appear

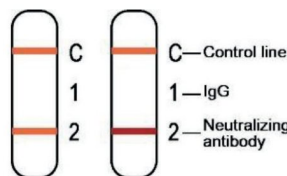

- **NEGATIVE**
→ Two lines appear

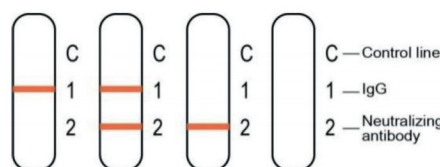

- **INVALID**
→ Control line fails to appear

Result determination time: The result should be judged within **25~30 minutes** after the sample is added into the sample loading well, and the result displayed after **30 minutes** is invalid.